

GLADIATOR MD™ VEST

Gladiator MD™ Vest

Instructions For Use

3724 Crescent Court West
Whitehall, PA 18052

610-215-2004

gladiatortherapeutics.com

*If you have any questions or concerns, please call us!
Use only as directed. Should you feel you are experiencing a medical
emergency, call 911.*





POWERED BY
SEMICERA®
TECHNOLOGY

Description

Gladiator Therapeutics Vest device uses patented SemiCera® Technology to reduce inflammation, and improve circulation in the body.

Indications

Gladiator MD Vest device may be worn on the torso area of the body to treat systemic inflammation or to improve circulation for conditions such as:

-  **Crohns Disease**
-  **Inflammatory Bowel Disease**
-  **Autoimmune Inflammation**
-  **Wound Healing**

Contraindication

Persons with Hemophilia should not use the device as it has not been thoroughly tested for this condition.

Pregnant women should only use under the supervision of a physician.

Use Recommendations

Do not leave device underneath any body part where the nodules may create a pressure point.

Device should not be used for a 48-hour period after surgery or if there is active bleeding from surgery or trauma.

Use of silver impregnated dressings will reduce efficacy of the device.

As treatment progresses, there may be increased sensitivity of the treatment area. Adjust device above or below the treatment area until it is comfortable for use.

Never place ice therapy on the device.

Maintain proper hydration when using the device.

Use of this device will not interact with any medication.

Application

- 1 Place arms through holes on both sides of vest with the opening in the front.
- 2 Fasten and adjust the three Velcro straps so that the device is snug but is not tight as to restrict circulation.
- 3 A thin layer of clothing such as a t-shirt is recommended under the vest to keep skin dry and protect from irritation.
- 4 The device can be utilized for as long as the user can support it for comfort and daily living activities.



A thin layer of clothing may be worn over the treatment area.

It is safe to utilize 24 hours a day.

Cleaning Instructions

The Gladiator device is designed to be patient dedicated. **The Gladiator device is designed to be patient dedicated. The device can be hand washed and hung dry if needed.**