

GLADIATOR[™] BLANKET MD[™]

Gladiator MD[™] Mini Blanket
& Gladiator MD[™] Large Blanket

Instructions For Use

3724 Crescent Court West
Whitehall, PA 18052

610-215-2004

gladiatortherapeutics.com

*If you have any questions or concerns, please call 1(800) IT-WORKS.
Use only as directed. Should you feel you are experiencing a medical
emergency, call 911.*

POWERED BY
SEMICERA[®]
TECHNOLOGY

Description

Gladiator Therapeutics Mini and Large devices use patented SemiCera® technology to reduce inflammation and improve circulation in the body.

Indications

Gladiator Therapeutics devices may be utilized on any area of the body that has inflammation or needs improved circulation. Some areas that have seen benefit from the SemiCera® technology are as follows:

-  **Pre-operative Treatment**
-  **Post-operative Treatment**
-  **Sprains & Strains**
-  **Systemic Wellness**
-  **Gastrointestinal Conditions**

Contraindication

Persons with Hemophilia should not use the device as it has not been thoroughly tested for this condition.

Pregnant women should only use under the supervision of a physician.

Use Recommendations

Do not leave device underneath any body part where the nodules may create a pressure point.

Device should not be used for a 48-hour period after surgery or if there is active bleeding from surgery or trauma.

Use of silver impregnated dressings will reduce efficacy of the device.

As treatment progresses, there may be increased sensitivity of the treatment area. Adjust device above or below the treatment area until it is comfortable for use.

Never place ice therapy on the device.

Maintain proper hydration when using the device.

Use of this device will not interact with any medication.

Application

- 1 The Gladiator Mini and Large devices can be used on surgical sites on any part of your body.
- 2 If you plan to be up and around, the device should be placed loosely over the treatment site and secured with the enclosed wrap.
- 3 Fasten and adjust the wrap so that the device stays in place but is not tight as to restrict circulation.
- 4 The longer you are able to wear the Gladiator the quicker you will heal or recover.
- 5 A minimum of 6 hours a day is the recommended amount of time to wear the device.



A thin layer of clothing may be worn over the treatment area.

It is safe to utilize 24 hours a day.

Cleaning Instructions

The Gladiator device is designed to be patient dedicated. **The device can be wiped down with an antimicrobial cleanser or disinfectant.**